

**Save time. Quickly twist and cut soft wire with control and ease.**

The gSource Wire Twister and Cutter is designed to provide exceptional holding, twisting and cutting ability.

- Twists and cuts cerclage wire from 0.8mm (21 gauge) up to 1.2mm (18 gauge).
- Custom wire twisters and cutters will be quoted for different OD wires based on supplied customer specifications.
- Expedites and uniformly twists cerclage wire.
- Knurled locking nut turns to unlock and release the handle so wire can be cut and ends clamped in jaws before twisting.
- Serrated 8mm jaws are designed to tightly hold wire when closed and in locked position.
- Knurled twisting knob on end helps to provide a secure grip as repeated pulling action is required until wire has reached desired tension.
- Grooved handles provide a secure grip when cutting or clamping.
- Made from German stainless steel.
- Quality verified and guaranteed by gSource.

**gS 82.4275** 10", max cap 1.2mm (18 gauge) soft wire

**Combination Wire Twister and Cutter**

For more information contact:

gSource, LLC  
 19 Bland Street  
 Emerson, NJ 07630

P (800) 978-1119  
 (201) 599-2277  
 F (201) 599-3306  
 E email@gSource.us  
 www.gSource.us

© gSource 2009  
 Printed in U.S.A.  
 wire twister and cutter OEM.idd.0809 01.1464.OEM

- How to use:
1. Wrap wire around bone using a gSource wire passer (gS 82.4800 - gS 82.4840) and position wire ends next to each other.
  2. Disengage knurled locking nut and grasp both ends of wire with jaw.
  3. Engage locking nut for hands free clamping.
  4. Stabilize the wiring site.
  5. Pull back on knurled twisting knob (repeatedly) until wire has reached desired tension.
  6. Disengage locking nut to release wire.
  7. Cut off wire ends.

